



QUICK GUIDE


eFOLDi 

Unfolding the eFOLDi Power Chair

Before starting place the Power Chair in an open space away from pets and small children.

Unfolding is straightforward and with a little practice you will be able to set it up in seconds.

We advise you to practise with the scooter before venturing out for the first time.

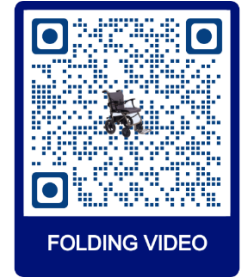

1 Anti-tip wheels and battery charger are supplied with your Power Chair.



2 Install the anti-tip wheels if desired. These simply press and lock into place.



3 Unfold the Power Chair by simply pushing the seat bottom down.



4 Keep controller cable away from pinch points like the armrest supports.



5 Install armrest supports into the slots and press them down until they lock.



6 Take the controller unit and..



7 ..insert the controller rod into the armrest fitting under the armrest..



8 ..adjust the position of the controller and tighten with the knurled knob.



9 Remove attendant control from back pocket and attach to the bar at rear.



10 Pull foot rest down.



11 Install seat cushions as shown. These are held in place by Velcro tabs..



12 ..make sure these are folded over the back rest and under the seat. Power Chair is now ready.

Folding the eFOLDi Power Chair



1 Slightly loosen the knurled knob and remove the controller.



2 Place controller and seatbelt strap onto seat.



3 Remove attendant control and place in pocket at rear of chair.



4 Ensure both armrests are set at their highest position..



5 - by pressing the small silver button to adjust armrests.



6 Check front wheels are positioned ahead to ensure smooth folding.
Tip pull the chair backwards to align the wheels



7 Then reach under edge of seat and pull the release bar up to release seat and..



8 ..pull seat upwards to fold the Power Chair.



9 Fold footrest in.

eFOLDi Power Chair Operation



Batteries can be removed by pressing the red tab upwards and pulling backwards.



You turn on main power here which powers handset controller.

Always turn this off when the chair is not in use to conserve battery power.



Turn the handset controller on.

The handset controller has buttons to adjust speed and displays to indicate power remaining and speed setting