



QUICK GUIDE

Unfolding the eFOLDi Power Chair

Before starting, place the Power Chair in an open space away from pets and small children.

Unfolding is straightforward and with a little practice you will be able to set it up in seconds.

We advise you to practise with the Power Chair before venturing out for the first time.



1 Anti-tip wheels and battery charger are supplied with your Power Chair.

2 Install the anti-tip wheels. These simply press and lock into place.

3 Unfold Power Chair by pushing the seat bottom down while pulling the release lever up.



4 Install armrest supports into the slots and press them down until they lock. Keep cables away from pinch points

5 Take the controller unit and rod and...

6 ...insert the controller rod into the armrest fitting under the armrest...



7 adjust the position of the controller and tighten with the knurled knob.

8 ... Remove attendant control from back pocket and attach to the bar at rear.

9 Pull foot rest down.



10 Install seat cushions as shown. These are held in place by Velcro tabs. Make sure these are folded over the backrest and under the seat.

11 Insert batteries until they click into place. Your Power Chair is now ready.

12 Turn on Power Chair by pressing the POWER button. Refer to your manual for full information.

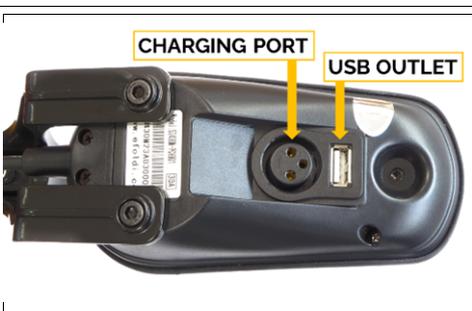
Folding the eFOLDi Power Chair

		
<p>1 Turn off power using the POWER button on the hand controller.. Always turn power off when the chair is not in use to conserve battery power.</p>	<p>2 Slightly loosen the knurled knob, remove controller. Place controller and seat belt on seat pad. Retighten the knob to prevent loss.</p>	<p>3 Remove attendant control and place it in pocket at rear of chair.</p>
		
<p>4 Check front wheels are positioned ahead to ensure smooth folding. Tip pull the chair backwards to align the wheels</p>	<p>5 Reach under edge of seat and pull the release bar up to release seat and...</p>	<p>6 ...pull seat upwards and towards you to fold the Power Chair. Fold in footrest.</p>

eFOLDi Power Chair Battery Charging

		
<p>To charge the Power Chair plug the charger lead into the base of the joystick controller and connect the AC plug to a power outlet. This will charge BOTH batteries simultaneously.</p>	<p>To remove batteries for travel or individual charging press the red tab upwards and slide the battery backwards.</p>	<p>Batteries may be charged separately from the Chair by using the short 'pigtail' adaptor which needs to be connected to the charger lead first.</p>

eFOLDi Power Chair Operation

		
<p>The handset controller has buttons to adjust speed and displays to indicate power remaining and speed setting.</p> <p>You can disable the reversing sounder by pressing the button indicated.</p>	<p>The charging port for your Power Chair is located under the handset for charging BOTH batteries simultaneously.</p> <p>There is also a USB outlet to power a phone or other USB device.</p>	<p>The attendant control can be set to adjust walking speed for an attendant/carer and also provides a battery check.</p> <p>The lever controls backwards or forwards motion.</p>