

## QUICK GUIDE Unfolding the eFOLDi Explorer Scooter

Before starting stand the scooter upright in an open space away from pets and small children.

Unfolding the scooter is straightforward, with a little practice you will be able to set it up in seconds.

We advise you to practise with the scooter before venturing out for the first time.



**1** Locate 'B' and 'C' latches.



**2** Press and hold 'B' and 'C' latches and...



**3** ...pull the drag/carry handle upright.



**4** Release the handlebar by pulling towards you.



**5** While pushing the handlebar down and inwards release the 'E' latch.



**6** Pull the steering column up and over the carry handle until it **locks** in place.



**7** Release the front wheel by a sharp pull or twisting.



**8** Lower the scooter to the ground holding carefully.



**9** Pull the steering column upright...



**10** ...and make sure the column key **locks** into place under latch 'A'.



**11** Pull the seat back upright.



**12** Push the footpegs down. Your scooter is now ready.

## Folding the eFOLDi Explorer Scooter

1	2	3
4	5	6
7	8	9
10	11	12

## The eFOLDi Explorer Battery/Charger

Remove the battery by unlocking using the key and applying gentle pressure. The battery has points for charging and for powering a mobile device.	Your charger is located in the base of one of the polystyrene liners.	Insert key and turn to power on go. <b>Recharge battery every 3 months or when power gauge shows 24 or less.</b>